



70 % (v/v) IPA ready to use solution for disinfecting surfaces.

## perform<sup>®</sup> classic alcohol IPA

### Our Plus

- fast, effective and residue free
- ready to use
- colour and perfume free
- efficacy tested according to European norms (EN)

### Application areas

For disinfection of surfaces in areas where fast microbiological effectiveness is required.

### Instructions for use

Apply perform<sup>®</sup> classic alcohol IPA undiluted to objects and surfaces and allow to take effect. Ensure complete wetting. The product dries off quickly. Maximum application rate 50 ml/sq.m. Do not treat surfaces that are particularly sensitive to alcohol such as acrylic glass.

### Microbiological efficacy

Efficacy	Concentration	Contact time
bactericidal <i>EN1276, EN13697</i>	ready-to-use	1 min.
yeastcidal <i>EN1650, EN13697</i>	ready-to-use	1 min.



# perform<sup>®</sup> classic alcohol IPA

## Product data

### Composition:

100 g solution contains the following active substances: 63,1 g Propan-2-ol (equates 70 % (v/v)).

### Chemical-physical data

Color	colourless
Density	approx. 0,88 g/cm <sup>3</sup> / 20 °C
Flash point	22 °C / Method : DIN 51755 Part 1
Form	liquid
pH	Not applicable
Viscosity, dynamic	> 2,5 mPa*s / 20 °C / Raw material

## Expert opinion and information

Please visit our website for an overview of all available literature/reports on the product: [www.schuelke.com](http://www.schuelke.com).

For individual questions:

Application Department

Phone: +49 40 52100-666

E-Mail: [info@schuelke.com](mailto:info@schuelke.com)

## Special advice

**Use disinfectants safely. Always read the label and product information before use.**

EU-0023656-0001 1-1

## Information for order

Item	Delivery form	Item no.
perform <sup>®</sup> classic alcohol IPA 1 l bottle	10/Carton	on request
perform cl. alc IPA EN/IT/E 1 l FL	10/Carton	on request
perform <sup>®</sup> classic alcohol IPA 10 l canister	1/Canister	on request

These products are not available in every country. For more information please contact our local subsidiary or distributor.

## Application aids

Application aids	Item no.
Can key for 5 + 10 l	135810
Compressed-air nozzle	120001
Decanter funnel	117901
schülke tap for 5 l / 10 l can	135501
Spray head for 1l bottle	180212

## Related Products

- perform<sup>®</sup> classic mix dry wipes

## Environmental information

schülke manufactures products economically and with advanced, safe and environmentally friendly production processes while at the same time maintaining our high quality standards.



Schülke & Mayr GmbH holds a Manufacturer's Authorisation according to sect 13 para 1 German Drug Law and Certificates of GMP Compliance for medicinal products.

**schülke Headquarters**  
Schülke & Mayr GmbH  
Robert-Koch-Str. 2  
22851 Norderstedt, Germany  
Phone +49 40 - 52100 - 0  
Fax +49 40 - 52100 - 318  
[www.schuelke.com](http://www.schuelke.com)  
[info@schuelke.com](mailto:info@schuelke.com)