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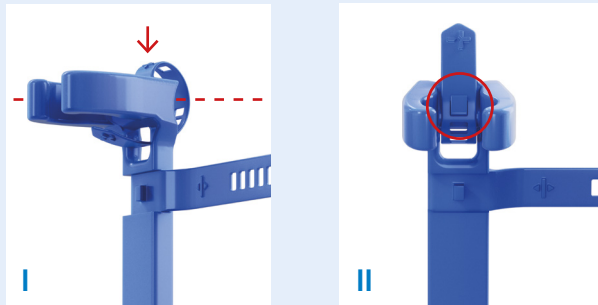
Point of Care (PoC) Halter 500 ml

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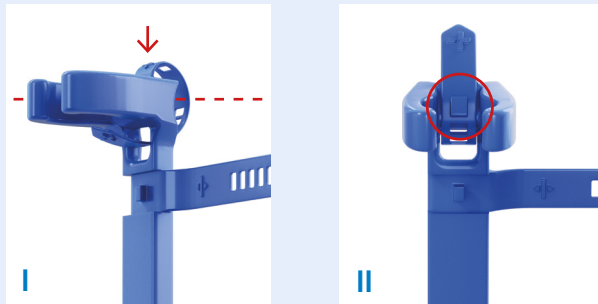
Manual

Point of Care (PoC) Holder 500 ml

1. Mounting on horizontal tubular frames

To attach the holder to a horizontal tubular frame, use the vertical fastening strap. The strap is provided with a user indication for horizontal fastening. Guide the strap from above around the tube bar and through the deflection eyelet (Fig. I). Push the upper bottom through the corresponding eyelet to close the strap (Fig. II). The strap should be pulled as tight as possible for maximum holding force.

Then bring the horizontal strap into its 'parking position': guide the strap behind the holder towards the front, then backwards through the deflection eyelet (Fig. III) and stow it at the rear side of the holder so that it does not interfere with the attachment (Fig. IV). Alternatively, the unused strap can be cut off. You can then insert the bottle.

3. Bottle size

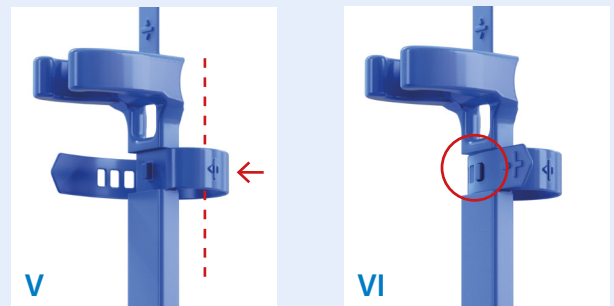
The PoC holder is suitable for holding 500 ml schülke bottles. Bottles from other manufacturers have not been tested for compatibility with the holder.

4. Maintenance

The PoC holder requires no maintenance.

2. Mounting on vertical tubular frames

To attach the holder to a vertical tubular frame, use the horizontal fastening strap. The strap is provided with a user indication for vertical fastening. Guide the horizontal strap around the tube bar (Fig. V) and press the lower button through the corresponding eyelet to close the strap (Fig. VI). The strap should be pulled as tight as possible for maximum holding force.

Then bring the vertical strap into its 'parking position': place it forwards and secure it through the eyelet provided on the head of the holder (Fig. VII). Alternatively, the unused strap can be cut off. You can then insert the bottle (Fig. VIII).

5. Care

Wipe down the PoC holder regularly with a soft disinfection wipe (e.g. mikroqid® universal wipes green line).

The PoC holder can be thermally reprocessed up to 60° C (e.g. with thermosept® X-tra enzymatic cleaner).