



Disinfectant foam cleaner based on alcohol, fruit acids and tensides.

## esemfix®

### Our Plus

- cleans and disinfects
- fresh fragrance
- excellent cleaning performance
- compatible with most materials
- leaves no streaks
- ready for use

### Application areas

Used for cleaning and disinfecting medical devices in all areas where cleanliness and hygiene is paramount, e.g.

- Treatment rooms
- Examination couches
- Work surfaces
- Reception areas
- Washrooms and social areas
- Laboratory equipment
- Dental spittoons

### Instructions for use

esemfix® comes in a 1 litre bottle with a special screw-in foam spray pump. Spray directly onto surfaces. Maximum quantity to be sprayed: 50 ml/m<sup>2</sup>. The foam makes it possible to see exactly which surfaces have been sprayed. Allow esemfix® to take effect and wipe off with a clean cloth. esemfix® is compatible with most materials. It leaves no streaks or visible residue. esemfix® is particularly useful in cleaning and disinfecting dental spittoons. Spray the spittoon completely with esemfix® and allow the active ingredients to take effect. Wipe around the spittoon with a disposable towel to remove any debris and calcium residue. Finally, rinse the spittoon with water. esemfix® does not interfere with the workings of the suction unit.

### Microbiological efficacy

Efficacy	Concentration	Contact time
<b>bactericidal</b> <i>in accordance with VAH Guidelines with short contact times</i>	ready-to-use	150 sec.
<b>fungicidal</b> <i>in accordance with VAH Guidelines with short contact times</i>	ready-to-use	150 sec.

CE 0297



## Product data

### Composition

100 g contains: 12.6 g Ethanol (94 %); 17.4 g Isopropyl alcohol;  
0.13 g Malic acid  
Other substances (in accordance with EC Regulation): < 5 %  
anionic tensides

## Special advice

**Use biocides safely. Always read the label and product information before use.**

For all of the related safety information please refer to the Safety Data Sheet. To request the Safety Data Sheet please E-mail: [tech.uk@schuelke.com](mailto:tech.uk@schuelke.com)

The product is highly compatible with metals and synthetics (except acrylic glass and alcohol-sensitive varnishes).

Wearing gloves is recommended. Do not use near ignition sources.

Please follow carefully any national fire and explosion protection guidelines when using alcohol-based disinfectants.

## Information for order

Item	Delivery form	Item no.
esemfix -INT- 1 l FL	10/Carton	183110

## Environmental information

schulke manufactures products economically and with advanced, safe and environmentally friendly production processes while at the same time maintaining our high quality standards.

## Expert opinion and information

Please visit our website for an overview of all available literature/reports on all schulke products online at <http://www.schuelke.com/>

For any queries you might have about the product or using it please call 0114 2543500 or E-mail [mail.uk@schuelke.com](mailto:mail.uk@schuelke.com)



Schülke & Mayr GmbH holds a Manufacturer's Authorisation according to sect 13 para 1 German Drug Law and Certificates of GMP Compliance for medicinal products.

**schulke UK**  
Schülke & Mayr UK Ltd  
Cygnet House, Meadowhall  
Sheffield, S9 1AT  
Phone +44-114-254 35 00  
Fax +44-114-254 35 01  
[mail.uk@schuelke.com](mailto:mail.uk@schuelke.com)

**schulke Headquarters**  
Schülke & Mayr GmbH  
Robert-Koch-Str. 2  
22851 Norderstedt, Germany  
Phone +49 (0) 40 - 52100 - 0  
Fax +49 (0) 40 - 52100 - 318  
[www.schuelke.com](http://www.schuelke.com)  
[mail@schuelke.com](mailto:mail@schuelke.com)