



Alcohol-based liquid for fast effective cleaning and disinfection of surfaces.

terralin[®] liquid

Our Plus

- rapid and effective in a variety of applications
- ready to use
- dries rapidly, leaving no streaks
- effective against bacteria (inc. TB), Fungi, Viruses (inc. HCV, Adeno, Rota, Papova, Polio virus)

Application areas

Alcohol-based rapid disinfection of medical devices in all areas with an increased risk of infection and where short contact times are required, e.g.:

- patient treatment units and adjoining surfaces
- examination tables
- operating tables and adjoining surfaces
- surfaces of medical equipment

Instructions for use

terralin[®] liquid is a ready-to-use solution. Apply the undiluted terralin[®] liquid onto objects and surfaces and allow it to take effect. Always aim for complete surface coverage. Do not exceed 50 ml/m². Make sure that all visible soiling is removed before disinfection and the lid is tightly closed after use. Do not treat surfaces that are sensitive to alcohol, e.g. acrylic glass. Not suitable for the disinfection of invasive medical devices.

Microbiological efficacy

Efficacy	Concentration	Contact time
bactericidal <i>in accordance with VAH Guidelines with short contact times</i>	ready-to-use	1 min.
MRSA	ready-to-use	1 min.
tuberculocidal	ready-to-use	1 min.

Efficacy	Concentration	Contact time
Mycobacterium terrae	ready-to-use	1 min.
yeasticial <i>in accordance with VAH Guidelines with short contact times</i>	ready-to-use	1 min.
virucidal against enveloped viruses <i>in accordance with DVV (German Registered Association for Combating Viral Diseases)//RKI Guideline</i>	ready-to-use	30 sec.
Vaccinia virus	ready-to-use	30 sec.
Norovirus <i>EN14476</i>	ready-to-use	1 min.
Poliovirus	ready-to-use	30 min.
Rotavirus	ready-to-use	30 sec.

CE 0297



Product data

Composition: 100 g solution contains the following active substances: 25 g Ethanol (94 %), 35 g 1-Propanol

Special advice

Use biocides safely. Always read the label and product information before use.

For all of the related safety information please refer to the Safety Data Sheet. To request the Safety Data Sheet please E-mail: tech.uk@schuelke.com

terralin[®] liquid is highly compatible with metals and synthetics (except acrylic glass and alcohol-sensitive varnishes).

Please follow carefully any national fire and explosion protection guidelines when using alcohol-based disinfectants. Store out of reach of children.

Information for order

Item	Delivery form	Item no.
terralin liquid 1 l FL	10/Carton	102001

Environmental information

Schülke manufactures products economically and with advanced, safe and environmentally friendly production processes while at the same time maintaining our high quality standards

Expert opinion and information

Please visit our website for an overview of all available literature/reports on all schulke products online at <http://www.schuelke.com/>

For any queries you might have about the product or using it please call 0114 2543500 or E-mail mail.uk@schuelke.com



Schülke & Mayr GmbH holds a Manufacturer's Authorisation according to sect 13 para 1 German Drug Law and Certificates of GMP Compliance for medicinal products.

schülke UK
Schülke & Mayr UK Ltd
Cygnet House, Meadowhall
Sheffield, S9 1AT
Phone +44-114-254 35 00
Fax +44-114-254 35 01
mail.uk@schuelke.com

schülke Headquarters
Schülke & Mayr GmbH
Robert-Koch-Str. 2
22851 Norderstedt, Germany
Phone +49 40 - 52100 - 0
Fax +49 40 - 52100 - 318
www.schuelke.com
info@schuelke.com