



Mildly alkaline liquid cleaner for the mechanical treatment of medical instruments and accessories.

## thermosept® alka clean forte

### Our Plus

- mildly alkaline instrument cleaner, not aggressive to materials and suitable for aluminium
- minimises risk of iatrogenic transfer of CJD/vCJD (acc. to RKI report)
- powerful cleaning with enzymes
- broad range of applications



### Application areas

thermosept® alka clean forte is particularly suitable for the treatment of surgical and microsurgical instruments, MIC instruments, rigid endoscopes, anaesthetic accessories and containers.

thermosept® alka clean forte is suitable for use in tunnel washers and individual machines.

### Instructions for use

- It is recommended that the water used for cleaning and the following program steps be at least softened and ideally demineralised (as for thermal disinfection).
- Other individually customisable program operations are available for the treatment of eye instruments or for program variations.
- thermosept® alka clean forte is suitable to be used for the cleaning step prior to thermal disinfection.
- Use in combination with thermosept® NKZ to ensure optimal treatment result. Neutralisation is not essential if water of suitable quality is used (i.e. RO water).

thermosept® alka clean forte is used in the following proportions:

Standard recommendations: 0.5% (5ml/l).

Depending on level of soiling: 0.3 - 1% (3 - 10ml/l) may be used.

The product is diluted by the machine's dosing pumps.



# thermosept® alka clean forte

## Product data

### Composition:

Labelling according to Regulation (EC) No. 648/2004: < 5 % nonionic surfactants, < 5 % anionic surfactants, < 5 % NTA and its salts, < 5 % polycarboxylates, enzymes.

## Special advice

Do not mix concentrate or working solution with acids. Keep container tightly closed. Protect from frost, heat and sunlight. Store at room temperature in the original container. For all of the related safety information please refer to the Safety Data Sheet. To view the Safety Data Sheet please visit [www.schuelke.com](http://www.schuelke.com)

## Information for order

Item	Delivery form	Item no.
thermosept alka clean forte 10 kg KA	1/Canister	147302

## Environmental information

schülke manufactures products economically and with advanced, safe and environmentally friendly production processes while at the same time maintaining our high quality standards.

## Expert opinion and information

Please visit our website for an overview of all available literature/reports on all schulke products online at <http://www.schuelke.com/>

For any queries you might have about the product or using it please call 0114 2543500 or E-mail [mail.uk@schuelke.com](mailto:mail.uk@schuelke.com)



Schülke & Mayr GmbH holds a Manufacturer's Authorisation according to sect 13 para 1 German Drug Law and Certificates of GMP Compliance for medicinal products.



**schülke UK**  
Schülke & Mayr UK Ltd  
Cygnet House, Meadowhall  
Sheffield, S9 1AT  
Phone +44-114-254 35 00  
Fax +44-114-254 35 01  
[mail.uk@schuelke.com](mailto:mail.uk@schuelke.com)

**schülke Headquarters**  
Schülke & Mayr GmbH  
Robert-Koch-Str. 2  
22851 Norderstedt, Germany  
Phone +49 40 - 52100 - 0  
Fax +49 40 - 52100 - 318  
[www.schuelke.com](http://www.schuelke.com)  
[info@schuelke.com](mailto:info@schuelke.com)