



Cleaning solution specially formulated for decontamination of dental instruments and hand pieces.

thermodent® alka clean

Our Plus

- to minimise the risk of iatrogenic transmission of CJD/ vCJD (in accordance with RKI notification)
- excellent material compatibility
- suitable for aluminium
- very good cleansing power
- good wetting properties due to the addition of surfactants
- lasts approximately 166 cycles

Application areas

thermodent® alka clean is suitable for the treatment of dental instruments, MIC instruments, containers, trays, handpieces and angle pieces.

Instructions for use

Application tips

The rinsing result is improved by using demineralized water and adding thermodont® clear to the final rinse.

Application methods

Following the cleaning step, it is advisable to rinse with thermodont® neutralizer to prevent the entrainment of alkali residues.

Use concentration:

thermodent® alka clean is used in the following proportions: Standard recommendation: 0.5 % (5 ml/l). Depending on level of soiling: 0.3 - 1 % (3 - 10 ml/l) may be used. The product is proportioned by the machine's own dosing pumps.



Product data

Concentration:

Labelling according to regulation (EC) No. 648/2004:

< 5 % nonionic surfactants, < 5 % anionic surfactants, < 5 % NTA and its salts, < 5 % polycarboxylates, enzymes, < 5 % NTA (nitrilotriacetic acid) and salts thereof.

Other ingredients: solubilizer, corrosion inhibitor.

Special advice

For all of the related safety information please refer to the Safety Data Sheet. To view the Safety Data Sheet please visit www.schuelke.com

Information for order

Item	Delivery form	Item no.
thermodent alka clean 5 l KA	1/Canister	147305

Environmental information

schülke manufactures products economically and with advanced, safe and environmentally friendly production processes while at the same time maintaining our high quality standards.

Expert opinion and information

Please visit our website for an overview of all available literature/reports on all schulke products online at <http://www.schuelke.com/>

For any queries you might have about the product or using it please call 0114 2543500 or E-mail mail.uk@schuelke.com



Schülke & Mayr GmbH holds a Manufacturer's Authorisation according to sect 13 para 1 German Drug Law and Certificates of GMP Compliance for medicinal products.

schülke UK
Schülke & Mayr UK Ltd
Cygnet House, Meadowhall
Sheffield, S9 1AT
Phone +44-114-254 35 00
Fax +44-114-254 35 01
mail.uk@schuelke.com

schülke Headquarters
Schülke & Mayr GmbH
Robert-Koch-Str. 2
22851 Norderstedt, Germany
Phone +49 40 - 52100 - 0
Fax +49 40 - 52100 - 318
www.schuelke.com
info@schuelke.com