



Ready to use impregnated wipes for disinfecting surfaces.

## perform® classic wipes EP

### Our Plus

- fast, effective and residue free
- economical with 200 jumbo wipes (approx. DIN A4 size) per box
- colour free and perfume free
- efficacy tested according to European norms (EN)

### Application areas

For disinfection of surfaces in areas where fast microbiological effectiveness is required.

### Instructions for use

Tear off wipes downward at an angle. Thoroughly wipe surfaces with perform® classic wipes EP, ensure complete wetting and allow to dry. The product dries off quickly. Close the lid tightly after use. Do not use on surfaces that are particularly sensitive to alcohol, such as acrylic glass.

### Microbiological efficacy

Efficacy	Concentration	Contact time
<b>bactericidal</b> <i>EN1276, EN13697</i>	ready-to-use	1 min.
<b>fungicidal</b> <i>EN1650, EN13697</i>	ready-to-use	2 min.
<b>virucidal against enveloped viruses</b> <i>in accordance with DVV (German Registered Association for Combating Viral Diseases)//RKI Guideline</i>	ready-to-use	30 sec.



# perform<sup>®</sup> classic wipes EP

## Product data

One box contains: 200 ready to use impregnated wipes.  
1100 g solution contains the following active substances:  
25 g Ethanol (94 %), 35 g Propan-1-ol.

## Special advice

**Use biocides safely. Always read the label and product information before use.**

For all of the related safety information please refer to the Safety Data Sheet. To request the Safety Data Sheet please

E-mail: [tech.uk@schuelke.com](mailto:tech.uk@schuelke.com)

IE: PCS Number 95503

## Information for order

Item	Delivery form	Item no.
perform classic wipes EP 200 ST DS	10/Carton	70000186

## Environmental information

schülke manufactures products economically and with advanced, safe and environmentally friendly production processes while at the same time maintaining out high quality standards.

## Expert opinion and information

Please visit our website for an overview of all available literature/reports on all schulke products online at <http://www.schuelke.com/>

For any queries you might have about the product or using it please call 0114 2543500 or E-mail [mail.uk@schuelke.com](mailto:mail.uk@schuelke.com)



Schülke & Mayr GmbH holds a Manufacturer's Authorisation according to sect 13 para 1 German Drug Law and Certificates of GMP Compliance for medicinal products.



**schülke UK**  
Schülke & Mayr UK Ltd  
Cygnet House, Meadowhall  
Sheffield, S9 1AT  
Phone +44-114-254 35 00  
Fax +44-114-254 35 01  
[mail.uk@schuelke.com](mailto:mail.uk@schuelke.com)

**schülke Headquarters**  
Schülke & Mayr GmbH  
Robert-Koch-Str. 2  
22851 Norderstedt, Germany  
Phone +49 40 - 52100 - 0  
Fax +49 40 - 52100 - 318  
[www.schuelke.com](http://www.schuelke.com)  
[info@schuelke.com](mailto:info@schuelke.com)