



Chlorine tablets with added surfactant for cleaning and degreasing surfaces, whilst providing a powerful disinfectant action.

with detergent

terralin[®] plus chlorine tablets

Our Plus

- effervescent chlorine releasing tablets
- added detergent for improved cleaning action
- broad spectrum of efficacy including bacteria, viruses, fungi and spores
- convenient and easy to use, just add water
- economic in use
- easy to store and distribute, no spillages or leaks
- unique lid offers additional safety in use

Application areas

Chlorine based preparation for the cleaning and disinfection of environmental areas, general disinfection and body fluid spills.

Instructions for use

Heavily soiled surfaces should be cleaned with a neutral detergent first. Add 1 tablet per 5 litres of water to make a solution of 200ppm if a more concentrated solution is required more tablets can be added each providing 200ppm. For terminal cleaning dissolve 1 tablet per litre of water to make a solution of 1000ppm. Wear protective clothing and gloves.

Application information

Available Chlorine (ppm)	Application	Tablets	Water	Other information
10,000	Body Fluid Spills	10	1L	2 mins contact time
2,500	Laboratory Jars	5	2L	Soak overnight
1,000	General Disinfection	1	1L	15 mins contact time
400	WC's, Drains, Sinks	2	5L	Pour in solution
200	Food Preparation Surfaces, Floors	1	5L	
60	Dishcloths, mops etc.	1	16L	



terralin[®] plus chlorine tablets

Product data

Compositions: Troclosene Sodium, Sodium Toulene Sulphonate, Sodium N-Lauroylsarcosinate. 53% NaDCC per tablet

Special advice

Use biocides safely. Always read the label and product information before use.

For all of the related safety information please refer to the Safety Data Sheet. To view the Safety Data Sheet please visit www.schuelke.com

Wear Protective clothing. Solution may bleach fabric. For professional use only. Contact with acids liberates toxic gas. Wash hands thoroughly after handling. Store in a well ventilated place. Keep container tightly closed. Dispose of contents/container in accordance with local regulations.

Order reference codes:

Cat. code: MRB1211

Information for order

Item	Delivery form	Item no.
terralin Plus chlorine Tablets 170 ST DS	12/Carton	70001644

Environmental information

schülke manufactures products economically and with advanced, safe and environmentally friendly production processes while at the same time maintaining our high quality standards.



Schülke & Mayr GmbH holds a Manufacturer's Authorisation according to sect 13 para 1 German Drug Law and Certificates of GMP Compliance for medicinal products.

schülke UK
Schülke & Mayr UK Ltd
Cygnets House, Meadowhall
Sheffield, S9 1AT
Phone +44-114-254 35 00
Fax +44-114-254 35 01
mail.uk@schuelke.com

schülke Headquarters
Schülke & Mayr GmbH
Robert-Koch-Str. 2
22851 Norderstedt, Germany
Phone +49 40 - 52100 - 0
Fax +49 40 - 52100 - 318
www.schuelke.com
info@schuelke.com