



Alkaline liquid cleaner for the automated reprocessing of medical devices and laboratory accessories.

## thermosept® alka clean forte

### Our Plus

- alkaline instrument cleaner with strong cleaning power
- minimizes risk of iatrogenic transfer of CJD/vCJD (acc. to RKI report)
- broad range of applications

### Application areas

thermosept® alka clean forte is particularly suitable for the efficient cleaning of surgical and microsurgical instruments, rigid endoscopes, anaesthetic accessories, containers and laboratory accessories. thermosept® alka clean forte is suitable for use in washers/ disinfectors and multichamber washers/ disinfectors.

### Instructions for use

Dosage:

Standard concentration: 0.5 % (5 ml/l) depending on the degree of soiling: 0.3 - 1.0 % (3 - 10 ml/l) possible.

Cleaning temperature standard: approx. 55°C (30 °C - 65 °C), cleaning time approx. 5 - 10 minutes. All parameters (dosage, temperature and time) depending on individual level of soiling. It is recommended that the water used for cleaning and the following program steps be at least softened and ideally demineralized (as for thermal disinfection).

Other individually customizable program operations are available for the treatment of eye instruments or for program variations. Combination with neutralizer thermosept® NKZ (or thermosept® NKP) ensures an optimal treatment result. Neutralization is not necessarily needed if demineralized water is used. Exclusively demineralized water should be used for thermal disinfection.

Please note recommendations of machine and instrument manufacturers.



## Product data

### Composition:

Labelling according to Regulation (EC) No. 648/2004: < 5 % nonionic surfactants, < 5 % anionic surfactants, < 5 % NTA and its salts, < 5 % polycarboxylates, enzymes.

### Chemical-physical data

Color	nearly colourless
Density	ca. 1,08 g/cm <sup>3</sup> / 20 °C / 1.013 hPa
Flash point	> 70 °C / Method : DIN 51755 Part 1
Form	liquid
pH	12 - 12,8 / 20 °C
Viscosity, dynamic	ca. 3 mPa*s / Method : ISO 3219

## Special advice

Do not mix up concentrate with other chemicals.  
Keep container tightly closed. Protect from frost, heat and sunlight. Store at room temperature in the original container.

## Information for order

Item	Delivery form	Item no.
thermosept alka clean forte 5 kg KA	1/Canister	on request
thermosept alka clean forte 10 kg KA	1/Canister	on request
thermosept alka clean forte 20 l KA	1/Canister	on request

These products are not available in every country. For more information please contact our local subsidiary or distributor.

## Application aids

Application aids	Item no.
Can key for 5 + 10 l	135810

## Related Products

- thermosept® NKP
- thermosept® NKZ

## Environmental information

schülke manufactures products economically and with advanced, safe and environmentally friendly production processes while at the same time maintaining out high quality standards.

## Expert opinion and information

Please visit our website for an overview of all available literature/reports on the product: <http://www.schuelke.com/>.  
For individual questions: Customer Sales Service Phone: +49 40 52100-666 E-Mail: [info@schuelke.com](mailto:info@schuelke.com)



Schülke & Mayr GmbH holds a Manufacturer's Authorisation according to sect 13 para 1 German Drug Law and Certificates of GMP Compliance for medicinal products.

**schülke Headquarters**  
Schülke & Mayr GmbH  
Robert-Koch-Str. 2  
22851 Norderstedt, Germany  
Phone +49 40 - 52100 - 0  
Fax +49 40 - 52100 - 318  
[www.schuelke.com](http://www.schuelke.com)  
[info@schuelke.com](mailto:info@schuelke.com)