



dosit<sup>®</sup> des

Dispenser for the production and application of aqueous disinfectant solutions.

## Our Plus

- Microprocessor-controlled system
- Monitoring of complete dispensing procedure by sensors
- Dispensing range: 0.25 % - 5 % in 0.05 % increments adjustable
- dispensing tolerance + 5 %
- Solution discharge depressurised directly at pivoting outlet
- Automatic switch-off at lack of concentrate or water as well as at disruption of concentrate flow
- Clear text display of all operating conditions incl. the adjusted concentrations
- Mains supply isolation according to DIN 1988, Part 4/TRWI) / EN 1717
- Built according to RKI guidelines RKI test under preparation

- The unit is built and tested according to all current regulations (VDE [German Association for Electrical, Electronic and Information Technologies], EMC [Electromagnetic compatibility], DVGW [German Association for Gas and Water], Robert Koch Institute [RKI], RKI test under preparation).



## Application areas

dosit<sup>®</sup> des is a microprocessor-controlled unit for the production of ready-to-use disinfectant solutions up to 5 %.

- All conventional surface and instrument disinfectants with a viscosity of < 60 mP•s can be used.
- The unit is designed for decentralized, stationary applications and discharges the solution for use depressurised.
- The operating conditions are shown on a 32-digit clear text display.
- Operation and programming takes place via a membrane keypad.
- The disinfectant concentrates can be adjusted in 0.05 % increments.
- The housing is made of stainless steel in a stylish bevelled finish. The suction lance and canister bracket can be mounted on the right or left of the unit. The canister console is easy to remove for cleaning purposes.
- The DVGW tested mains supply isolation enables its direct connection to a drinking water connection.



## Product data

### Dimensions | Technical data

width x depth x height: 51 x 18 x 57.8 cm

dispensing amounts: 7 litres/minute

supply voltage: AC 230 V - 50 Hz

power: 75 Watt

water connection: 3/8" external screw thread

## Information for order

Item	Delivery form	Item no.
dosing devices dosit <sup>®</sup> des	1/Carton	on request

These products are not available in every country. For more information please contact our local subsidiary or distributor.

## Related Products

- dosing unit D
- dosing unit Duo

## Environmental information

schülke manufactures products economically and with advanced, safe and environmentally friendly production processes while at the same time maintaining out high quality standards.

## Expert opinion and information

Please visit our website for an overview of all available literature/reports on the product: [www.schuelke.com](http://www.schuelke.com).

For individual questions:

Customer Sales Service

Phone: +49 40 52100-666

E-Mail: [info@schuelke.com](mailto:info@schuelke.com)



Schülke & Mayr GmbH holds a Manufacturer's Authorisation according to sect 13 para 1 German Drug Law and Certificates of GMP Compliance for medicinal products.

schülke Headquarters  
Schülke & Mayr GmbH  
Robert-Koch-Str. 2  
22851 Norderstedt, Germany  
Phone +49 40 - 52100 - 0  
Fax +49 40 - 52100 - 318  
[www.schuelke.com](http://www.schuelke.com)  
[info@schuelke.com](mailto:info@schuelke.com)