



Universal alkaline cleaner for automated processing of medical and laboratory instruments, glassware and utensils

mucapur® MF

Our Plus

- effective cleaning, removes contaminants such as blood, protein, tissue residues and starchy material as well as coffee and tea stains
- ideal for cleaning laboratory glassware and laboratory utensils
- very gentle, low-foaming formulation
- chelating agents protect machine parts from scale build-up

Application areas

mucapur® MF is used for thorough and gentle cleaning of medical instruments and laboratory utensils made of materials such as glass, porcelain, ceramic, plastic, quartz, and metal. The concentrate is suitable for use in washer-disinfectors (WDs) and discontinuous belt washing systems.

Instructions for use

Standard concentration 0.1 % to 0.5 % (1 ml/l - 5 ml/l), time approx. 5 - 10 min., temperature approx. 40 - 65 °C. Concentration, temperature and contact time depending on the degree of contamination.

Do not use mucapur® MF in combination with other products. The use of deionised water is recommended. Dosage is performed automatically by the machine's metering pumps. Please note the manufacturers' recommendations for devices and instruments.



Product data

Composition:

Labelling according to Regulation (EC) No. 648/2004: 5 % - 15 % phosphate, < 5 % polycarboxylate, < 5 % chlorine-based bleaching agents.

Further ingredients: alkalines, silicates, complexing agents, corrosion inhibitors.

Chemical-physical data

Color	light yellow
Density	ca. 1,30 g/cm ³ / 20 °C
Flash point	Not applicable
Form	liquid
pH	> 12 / 20 °C / concentrate
Viscosity, dynamic	No data available

Special advice

Contains Potassium hydroxide.

Information for order

Item	Delivery form	Item no.
mucapur-MF 5 l KA	1/Canister	on request
mucapur MF 20 l KA	1/Canister	on request
mucapur MF 10 l KA	1/Canister	on request

These products are not available in every country. For more information please contact our local subsidiary or distributor.

Accessories

Accessories	Item no.
Can key for 5 + 10 l	135810

Environmental information

schülke manufactures products economically and with advanced, safe and environmentally friendly production processes while at the same time maintaining out high quality standards.

Expert opinion and information

Please visit our website for an overview of all available literature/reports on the product: www.schuelke.com. For individual questions: Customer Care Phone: +49 40 52100-666 E-Mail: info@schuelke.com



Schülke & Mayr GmbH holds a Manufacturer's Authorisation according to sect 13 para 1 German Drug Law and Certificates of GMP Compliance for medicinal products.



schülke Headquarters
Schülke & Mayr GmbH
Robert-Koch-Str. 2
22851 Norderstedt, Germany
Phone +49 (0) 40 - 52100 - 0
Fax +49 (0) 40 - 52100 - 318
www.schuelke.com
mail@schuelke.com

Schülke & Mayr UK Ltd.
1 Jenkin Road
GB-Sheffield S9 1 AT
Phone +44 (0) 1142 - 5435 - 00
Fax +44 (0) 1142 - 5435 - 01
www.schuelke.co.uk
mail.uk@schuelke.com

Schülke & Mayr Ges.m.b.H.
Seidengasse 9
1070 Wien, Austria
Phone +43 (0) 1 - 5232501-0
Fax +43 (0) 1 - 5232501-60
www.schuelke.at
office.austria@schuelke.com